ADVANCED SILICONE SCAR SHEET

D.G.DENA Co.,
Novara Ave., Industrial Zone,
Shiraz,
ESKA Diagnostics Kft.,
1073 Budapest, Hungary
www.dgdena.com

FR

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EN

Name
DG.Derm silicone sheets

Product Description
DG.Derm silicone sheets are a proven technology for treating scars (hypertrophic and keloid) and preventing abnormal scar development after surgery or an accident. The medical silicone sheet is self-adhesive, washable and can be used for a long period of time.

Composition
Medical grade silicones

Properties
DG.Derm silicone sheets ensure that scars become flatter, smoother and thinner and reduce complaints (like persistent redness, pulling of the scar itching, restriction of joint movement, discomfort and cosmetic concerns).

Indications/applications
DG.Derm silicone sheets are suitable for:
• Old and new scars
• All skin types
• All ages
• Treatment and prevention of abnormal scars (hypertrophic and keloid)
• Treatment of burn scars
• Treatment of surgical scars

- Instructions for use**
- For optimum results preferably start using DG.Derm silicone therapy immediately after wound closure (after removal of sutures and the disappearance of any scab).
 - It is recommended that silicone therapy is built gradually, up to the maximum of 23 hours use per day. The purpose of building up gradually is to allow the skin to become accustomed to the silicone.
 - For the best possible results, it is preferable that the silicone sheets are used for a minimum of 12 and a maximum of 25 hours per day.
 - If necessary, DG.Derm silicone sheets can be cut to the correct size.
 - Ensure that the silicone sheet extends two centimetres beyond the edges of the scar.
 - Ensure good adhesion and durability of the silicone sheet by removing any hair around the scar site.
 - Thoroughly clean and dry the skin before applying the silicone sheet to the scar and the area surrounding it.
 - Remove the protective film from the back of the silicone sheet and apply DG.Derm to the scar.
 - If necessary, the silicone sheet can be fixed in place with an adhesive or elastic bandage.
 - After removal of the silicone sheet, the skin may be treated with a hydrating cream.

Care instructions for DG.Derm silicone sheets

- It is essential to clean the DG.Derm silicone sheets properly, on a daily basis. This is necessary to ensure good adhesion.
- Rub the silicone sheet with a few drops of neutral soap.
- Remove as much dirt as possible with soap and lukewarm water.
- Rinse the silicone sheet thoroughly under lukewarm running water.
- Leave the silicone sheet to dry on a clean, dry surface, with the sticky side facing upwards.
- Use the silicone sheet only after it has completely dried.

How long should I use DG.Derm silicone sheets?

- It takes at least 3-6 months for a scar to normalise.
- Use the DG.Derm silicone sheet as long as the scar is raised, red, or causes discomfort.
- Consult your physician about the duration of the therapy.

Special attention and precautions

- Keep out of reach of children.
- DG.Derm silicone sheets are only suitable for external use.
- Use DG.Derm silicone sheets only on healed, closed skin (after the scab has disappeared) and never on open wounds.
- Do not dry DG.Derm silicone sheets in direct sunlight or near a heat source.
- Temporarily remove the DG.Derm silicone sheet when exercising or swimming.
- Contact with water may cause the DG.Derm silicone sheet to detach.
- Do not use if you are hypersensitive to any of the ingredients.

Side effects

- In rare cases, the skin may have a hypersensitive reaction (skin irritation) to silicone treatment. In such cases, stop the treatment until the skin is no longer irritated.
- Start the treatment again and let the skin slowly get used to the DG.Derm silicone sheet (applying it for an additional hour every day).
- If skin problems persist, consult your physician.

Storage

- The expiry date is stated on the packaging.
- DG.Derm silicone sheets can be stored for 3 years in its unopened packaging.
- Store at temperatures between 5°C and 25°C.

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